Instructions for use. For information only.
Please read carefully and retain for future reference.

MeniCare Plus

MULTIPURPOSE SOLUTION
For all rigid gas permeable lenses

INDICATIONS

- CLEANING
- RINSING after cleaning
- DISINFECTING : effective within 5 minutes
- SOAKING
- RINSING before insertion and LUBRICATING

DIRECTIONS

- Wash your hands thoroughly before handling your contact lenses :
- Rub the lens carefully with several drops of MeniCare Plus solution during 20 seconds.
 - Place the lenses in the Right and Left lensholder and rinse them with MeniCare Plus.
 - Fill the lens case with MeniCare Plus and soak the lenses for a minimum of 5 minutes or overnight.
 - Before insertion, rinse the lenses with MeniCare Plus.

WARNINGS

- Renew the solution in the lens case after each use and at least once a month in case of extended soaking periods due to lens wear interruption.
- Keep out of children's reach.
- Never use tap water or non-sterile water with contact lenses as microbial contamination can occur, which may lead to permanent eye damage.
- Consult your doctor or eye care professional before using any eye medication with contact lenses.

PRECAUTIONS

- Once opened, use MeniCare Plus within 3 months and make sure that the bottle is tightly closed after each use.
- Do not touch the bottle top with fingers.
- Store at room temperature ($\leq 25^{\circ}\text{C}$).
- Observe the expiry date printed on the box and the bottle.
- Not to initially use the product, if the safety ring is damaged.
- Do not swallow.
- Clean the lens case regularly and renew it every 3 months.
- In case of irritation, remove the lenses and consult your eye care practitioner.
- Store out of sunlight

CONTRA-INDICATIONS

- Do not use with soft contact lenses.
- Do not use in case of allergic background to one of ingredients.

ADVERSE REACTIONS

If you experience persistent discomfort or irritation, remove your lenses, stop using the product and consult your eye care professional.

CONTENTS

Polyhexamethylene Biguanide	0.0005 %
Poloxamer	0.5 %
Hypromellose	0.275 %

PACKAGING

Menicare Plus is available in 250 ml, 125 ml and 50 ml bottles.

Manufacturer / Hersteller / Fabrikant :
Menicon Pharma
Parc d'Innovation - 67400 Illkirch Grafenstaden - France

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These instructions contain important information.
Please read carefully and retain for future reference.

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